GREEN WOUND HEALING

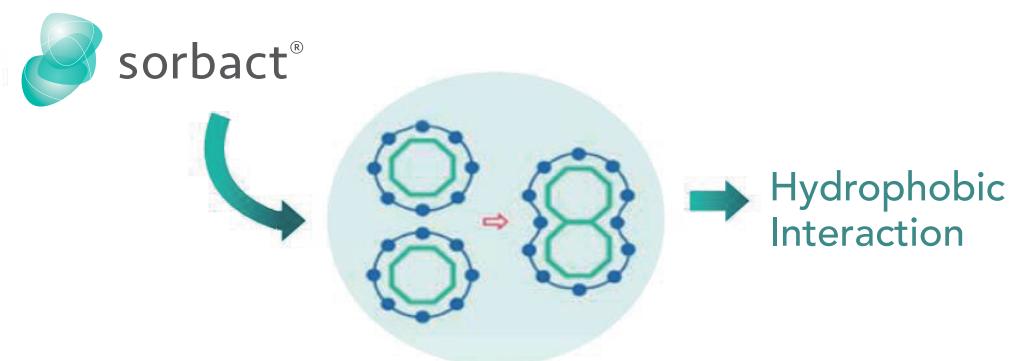
WOUND HEALING ALL OVER THE WORLD **WITH A GLOBAL COMMITMENT**

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Hydrophobicity in some of the most common pathogenic microorganisms

■ Enterococci 94%

■ Staph.aureus 90%

■ E.coli 85%

■ Candida albicans 84%

OUR SOLUTION

SORBACT® SAFELY REDUCES THE BIOBURDEN IN WOUNDS

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No active agents are released into the wound

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Binds common microbes in the wound – as well as multi-resistant bacteria

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INCREASING MICROBIAL VIRULENCE AND/OR NUMBERS

CLEAN

CONTAMINATION

COLONIZATION

LOCAL INFECTION
Not necessarily clinical signs

SPREADING INFECTION

SYSTEMIC INFECTION

SORBACT® PREVENTS AND TREATS WOUND INFECTIONS BY REDUCING THE WOUND BIOBURDEN

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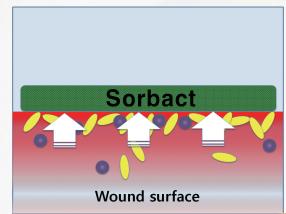
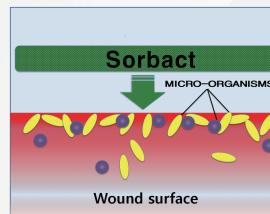
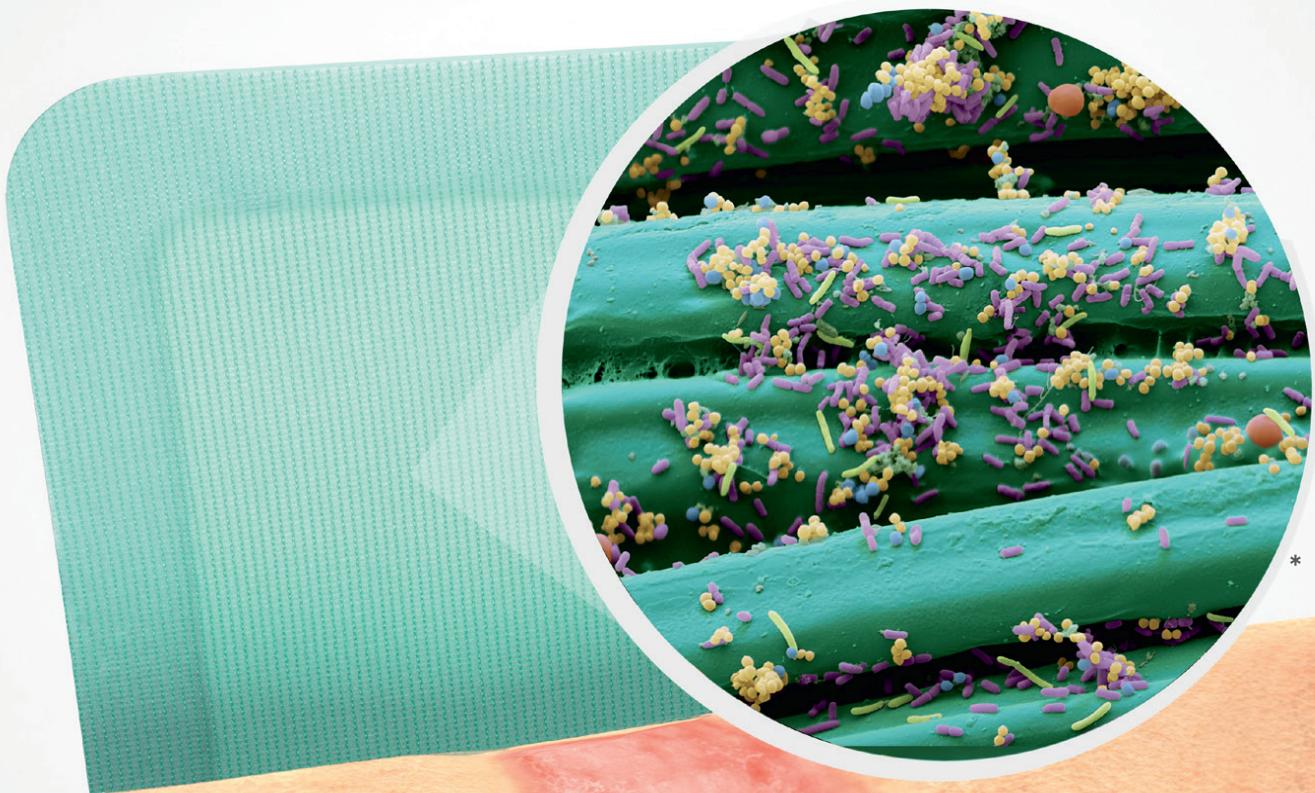
MAKES SENSE

SORBACT® ADVANTAGES OVER COMMON ANTIMICROBIALS

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	 sorbact®	COMMON ANTIMICROBIAL DRESSINGS
CAN BE USED TO PREVENT WOUND INFECTION - WITHOUT RESTRICTIONS	✓	✗
BINDS AND REMOVES MICROBES FROM THE WOUND	✓	✗
REDUCES THE BIOBURDEN IN WOUNDS	✓	✓
IMMEDIATE ONSET OF ACTION	✓	✓
NO MECHANISM OF RESISTANCE HAS BEEN DESCRIBED	✓	✗
NO RELEASE OF ACTIVE AGENTS INTO THE WOUND	✓	✗

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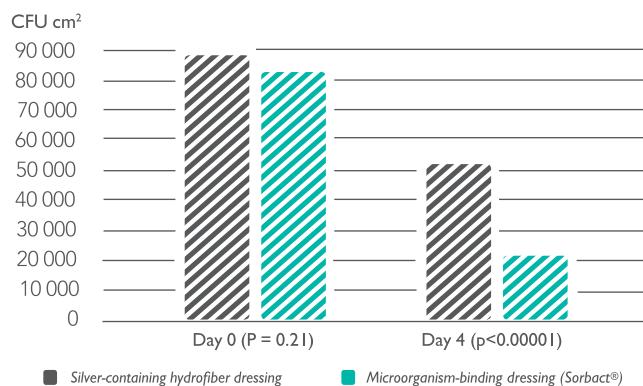
Sorbact® significantly reduces bacterial burden in infected leg ulcers compared to a silver-containing dressing

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
Sorbact® is proven effective in the management of chronic wounds

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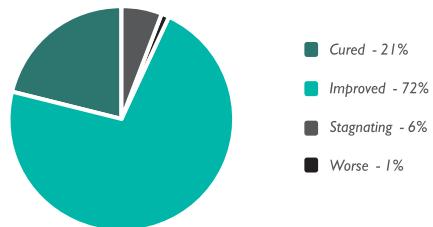
FINAL OUTCOMES OF THE WOUND-HEALING PROCESS

Sorbact® - the cost-effective way to prevent surgical site infection

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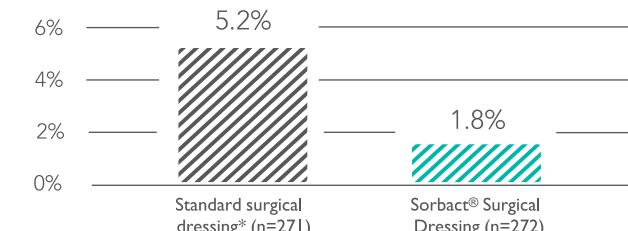% SURGICAL SITE INFECTION (SSI)

TOTAL ESTIMATED COST

The SSI rate in the Sorbact® Surgical Dressing group was 1.8%, which is significantly lower than the 5.2% in the control group ($p=0.04$). The total estimated cost of SSI prophylaxis and treatment was higher in the control group compared to the Sorbact® Surgical Dressing group (€5 775 vs €1 065, respectively)⁸⁾

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 Dry
 Low
 Medium
 High

 Superficial
 Shallow
 Shallow + deep

 Infection prevention
 Sloughy
 Infected
 Necrotic

SORBACT® GEL DRESSING

Area of use
For dry to low-exuding wounds, also burns.
Adds moisture.

SORBACT® COMPRESS

Area of use
For superficial and cavity wounds.

POST OPERATIVE				
CLEAN				
COLONIZED / INFECTED				
NECROTIC / DRY FIBRINOUS				
SUPERFICIAL				
CAVITIES				
FISTULAS				
NPWT				
FUNGAL INFECTION IN SKIN FOLDS				

**SORBACT®
SURGICAL
DRESSING**

💧 — 💧

Area of use
For infection control in post-surgical and traumatic wounds.
Shower proof backing film.

**SORBACT®
RIBBON GAUZE**

⬇️ 💧

Area of use
For fistulas and fungal infection in skin folds.

**SORBACT®
ROUND SWAB**

⬇️ 💧

Area of use
For cavity wounds and cleansing.

**SORBACT®
F - DRESSING**

💧 ⬇️

Area of use
For low to moderately exuding wounds.
Semi-permeable backing film.

**SORBACT®
ABSORPTION
DRESSING**

💧 ⬇️

Area of use
For moderately to highly exuding wounds.

**SORBACT®
SUPER-
ABSORBENT**

💧 ⬇️

Area of use
For highly to excessively exuding wounds.

REFERENCE

1. Ljungh Å, Hjerten S, Wadström T. High surface hydrophobicity of aggregating *Staphylococcus aureus* strains isolated from human infections studied with the salt aggregation test, SAT. *Infect Immun* 1985;47:522-6
2. Ljungh Åm, Öterlind M, Wadström T. Cell surface hydrophobicity of Group D and viridans streptococci isolated from patients with septicaemia. *WBI Bakteriol Mikrobiol Hyg* 1986;A261:280-6
3. Doyle RJ, Rosenberg M(eds). *Microbial cell surface hydrophobicity*. ASM, Washington DC, 1990
4. Ronner AC, et al. Adhesion of methicillin-resistant *Staphylococcus aureus* to DACC-coated dressings. *J Wound care* 2014; 23(10):484, 486-488
5. Ljungh A, et al. Using the principle of hydrophobic interaction to bind and remove wound bacteria. *J Wound Care* 2006;15(4):175-180
6. Mosti G, et al. Comparative study of two antimicrobial dressings in infected leg ulcers: a pilot study. *J Wound care* 2015;24(3):121-122
7. Kammerlander G, et al. An investigation of Cutimed® Sorbact® as an antimicrobial alternative in wound management. *Wounds UK* 2008;4(2):10-18
8. Stanirowski PJ, et al. Randomized Controlled Trial Evaluating Dialkylcarbamoyl Chloride Impregnated Dressings for the Prevention of Surgical Site Infections in Adult Women Undergoing Cesarean Section. *Surg Infect (Larchmt)* 2016; 17(4):427-35

ABIGO Medical AB
Ekonomivägen 5 · SE-436 33 Askim · Sweden
Tel: +46 31-748 49 50 www.abigo.com

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